






- 1** Download the **Karie Companion App** onto your mobile device from Google Play or the App Store

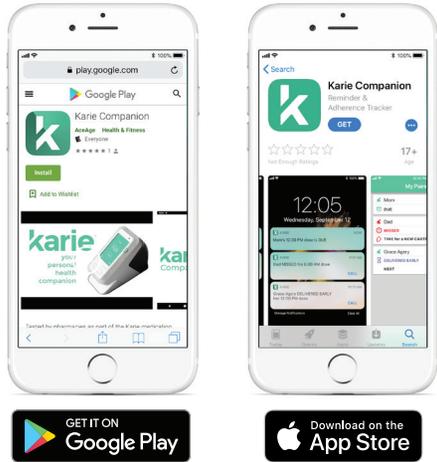

- 2a** Launch the App and set up your account. Enter your first name, last name and email address

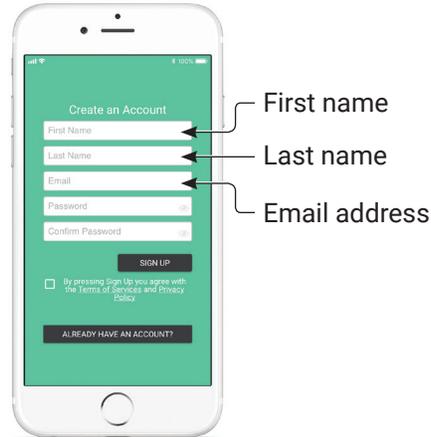


- 2b** Enter a password with these rules:
1. At least 8 characters
 2. At least 1 Upper Case letter
 3. At least 1 lower case letter
 4. At least one number, and
 5. At least one special character, like: ! @ #

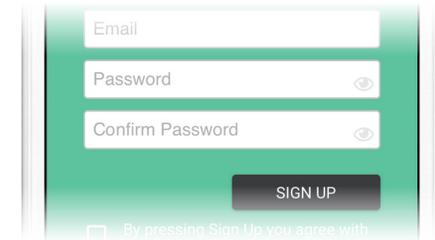


Keep your password somewhere safe


- 3** **Check your email:** The App will send an email to the address you provided. Click the green button to confirm your account

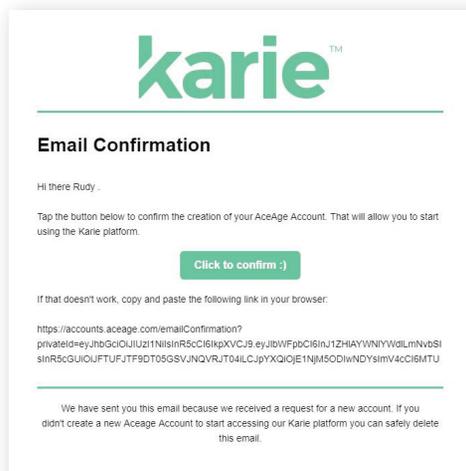

- 4** **My Pairings** screen: Press the ADD (+) button in the lower right corner to begin pairing. *You must have access to the Karie device*

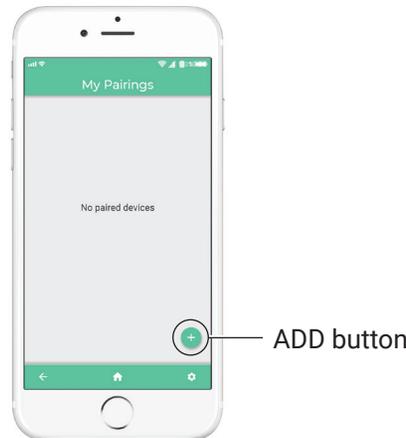

- 5** **Pair Karie** screen: You can scan a QR code **OR** enter a PIN when it is generated by the Karie device

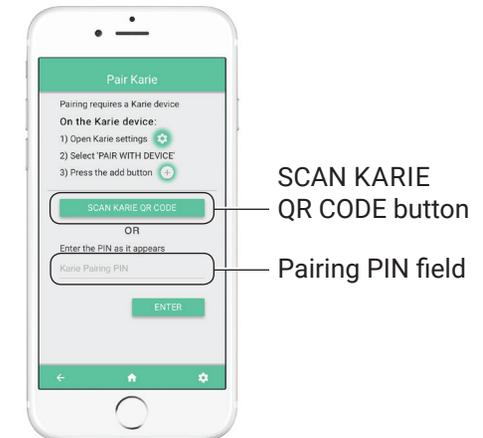



6 On the **Karie device**: On the Home screen, press the **SETTINGS** icon in the upper right corner




7 On the **Karie** Settings screen, press the **PAIR WITH DEVICE** button




8 **Karie** will display a QR code and a PIN. They are temporary and expire after 2 minutes



9 If they expire, **Karie** will prompt you to re-generate a new QR code and PIN

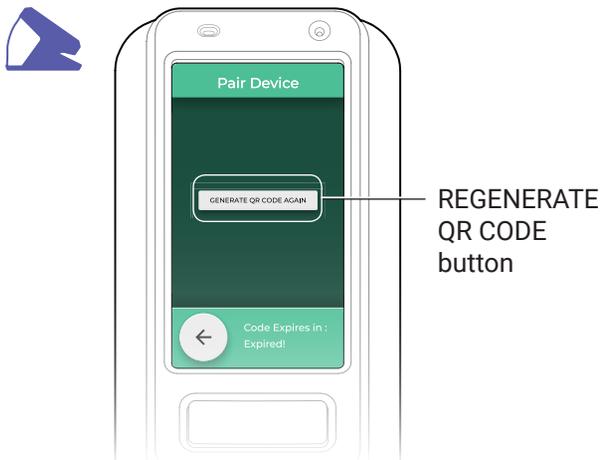

10a On your mobile device: Press **SCAN KARIE QR CODE** in the App and hold your device above Karie to scan the code

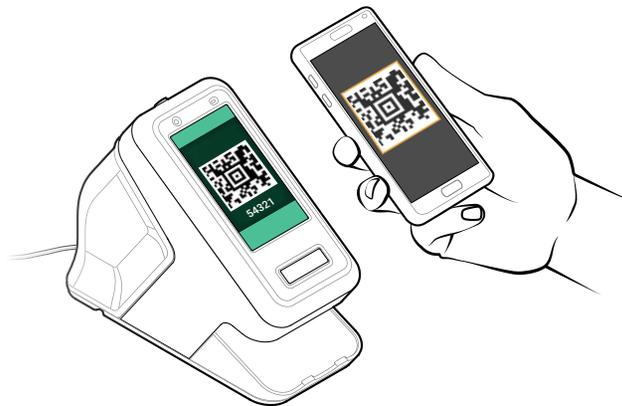


10b Alternatively, you can enter the PIN that appears underneath the QR code






11 Once paired, select whether you are the user of the Karie device or a caregiver for that user

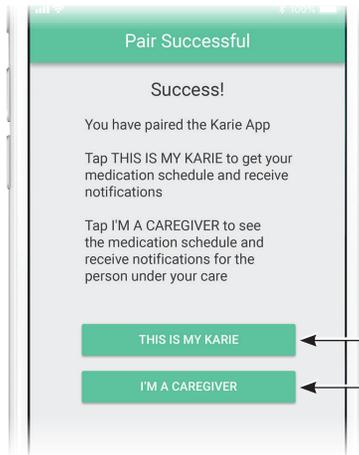


THIS IS MY KARIE
I'M A CAREGIVER

12 You will now see the paired Karie user on the My Pairings screen

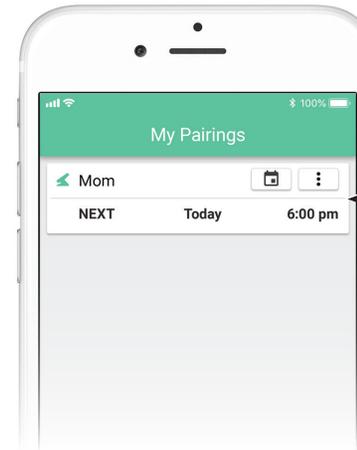

Paired Karie user


13 Press the Karie user's SCHEDULE (📅) button to review the schedule for the doses in their current cartridge

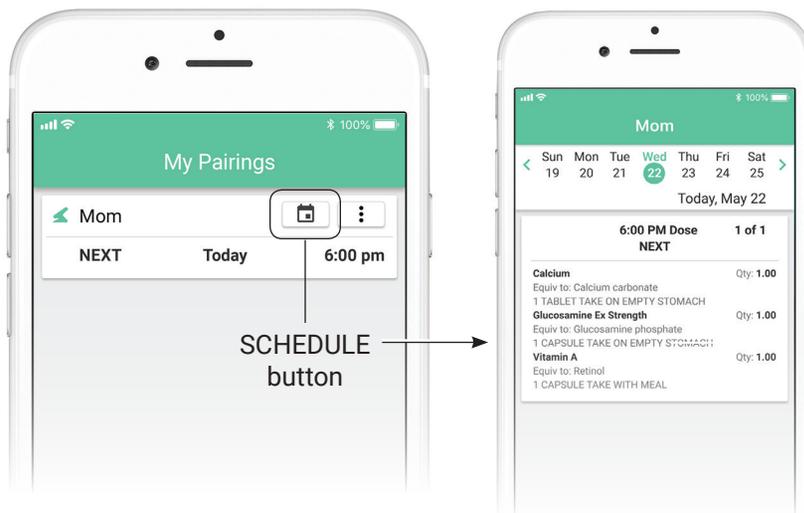

SCHEDULE button

14 Press the Karie user's MORE (⋮) button to:

1. Specify which notifications to receive
2. Set the warning window for Early and Missed Doses, and
3. Enter the Karie user's phone number to call directly from the App

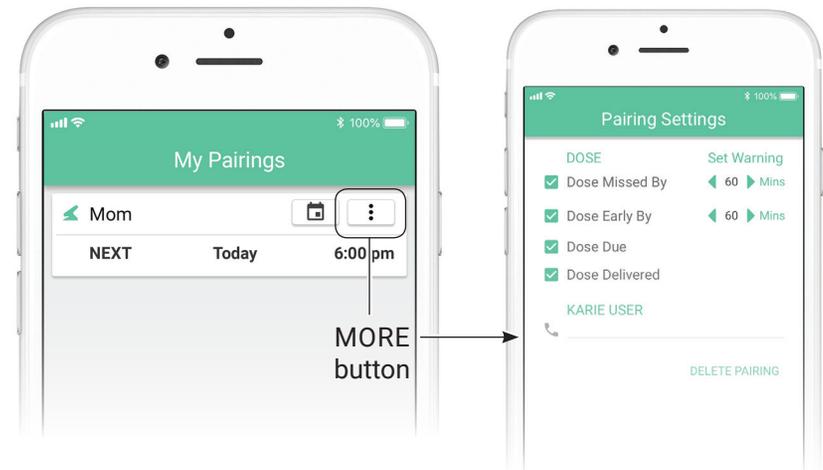

MORE button